EN

COSMEDENT® www.cosmedent.com | 800.621.6729
Enamelize® | A COMPOSITE POLISHING PASTE

INSTRUCTIONS FOR USE

DESCRIPTION

Enamelize® The final polish! **Enamelize** will bring natural tooth structure, dental composite resin, porcelain and metals to the highest polish attainable. This dental polishing paste will greatly increase the gloss, luster and surface smoothness of your restoration. Use **Enamelize** as the final polish on microfills, nanofills, microhybrids, metals and natural dentition. It is also the best material for the routine care of bonded restorations. **Enamelize** creates a shine that is so beautiful you must try it to believe it.

COMPOSITION

Enamelize consists of water, aluminum oxide, emulsifier, glycerin, vegetable oil and methylparaben.

INDICATIONS FOR USE

- Polishing of composite materials.
- Polishing of all metal restorations.
- After diamond polishing paste on porcelain.
- After use of prophy paste on natural dentition.

DISPOSAL INSTRUCTIONS

Dispose of contents/ container in accordance with local regulation.

CONTRAINDICATIONS

Enamelize is contraindicated if patient is known to be allergic to any of the ingredients of **Enamelize**.

SIDE EFFECTS

May cause irritation in sensitive patients.

INTERACTIONS

None

DIRECTIONS FOR USE

- ① Apply **Enamelize** to areas you wish to polish.
- ② Light pressure buffing with a rubber prophy cup or felt wheel is all that is needed to finish with a sheen that will amaze you and will enhance the quality of your restoration.
- ③ A soft buff with a dry piece of gauze will increase the luster.

HYGIENE/DISINFECTION

Disinfection of the syringe can be carried out using commercially available disinfecting solution.

NOTICE

Serious incidents must be reported to DeltaMed GmbH, the responsible importer and the responsible authority

WARNING

Enamelize may cause irritation in sensitive persons. In case of contact with the eyes, rinse immediately with plenty of water and seek medical attention.

STORAGE

- Store at 39-75°F (4 - 24°C).
- Shelf life: see information on syringes and/or packages.
- Keep away from sunlight.
- Keep material out of reach of children.
- Do not use after the expiration date.

DISPOSAL

Do not allow to enter into surface water or drains. Dispose of waste according to applicable legislation.

PACKAGING

Enamelize 3 gram individual syringe, **Enamelize** 36 gram tube

Rx **IMPORTANT PRODUCT INFORMATION**

Federal (U.S.) law restricts this device to sale by or on the order of a dentist.

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Cosmedent® Inc.
Se deben informar los incidentes graves a DeltaMed GmbH, el importador responsable y la autoridad responsable.