

**COSMEDENT®**

Enamelize®

A Composite Polishing Paste

## INSTRUCTIONS FOR USE

**www.cosmedent.com**  
**800.621.6729**

**DESCRIPTION**

**Enamelize®** The final polish! **Enamelize** will bring natural tooth structure, dental composite resin, porcelain and metals to the highest polish attainable. This dental polishing paste will greatly increase the gloss, luster and surface smoothness of your restoration. Use **Enamelize** as the final polish on microfills, nanofills, microhybrids, metals and natural dentition. It is also the best material for the routine care of bonded restorations. **Enamelize** creates a shine that is so beautiful you must try it to believe it.

**COMPOSITION**

**Enamelize** consists of water, glycerin, vegetable oil, emulsifier and aluminum oxide.

**INDICATIONS FOR USE**

- Polishing of composite materials.
- Polishing of all metal restorations.
- After diamond polishing paste on porcelain.
- After use of prophylaxis paste on natural dentition.

**CONTRAINDICATIONS**

**Enamelize** is contraindicated if patient is known to be allergic to any of the ingredients of

**Enamelize.****SIDE EFFECTS**

May cause irritation in sensitive patients.

**INTERACTIONS**

None.

**DIRECTIONS FOR USE**

- 1** Apply **Enamelize** to areas you wish to polish.
- 2** Light pressure buffing with a rubber prophylaxis cup or felt wheel is all that is needed to finish with a sheen that will amaze you and will enhance the quality of your restoration.
- 3** A soft buff with a dry piece of gauze will increase the luster.

**HYGIENE**

Disinfection of the syringe can be carried out using commercially available disinfecting solution.

**WARNING**

**Enamelize** may cause irritation in sensitive persons. In case of contact with the eyes, rinse immediately with plenty of water and seek medical attention.

**STORAGE**

- Store at 39-75° F (4 - 24°C).
- Shelf life: see information on syringes and/or packages.
- Protect **Enamelize** from heat and sunlight.
- Keep material out of reach of children.
- Do not use after the expiration date.

**DISPOSAL**

Do not allow to enter into surface water or drains. Dispose of waste according to applicable legislation.

**PACKAGING**

**Enamelize** 3 gram individual syringe,  
**Enamelize** 36 gram tube


**R<sub>x</sub> IMPORTANT  
PRODUCT  
INFORMATION**

**Federal (U.S.) law restricts this device to sale by or on the order of a dentist.**

**Note:** This product was developed for use in the dental sector and should be used in accordance with the instructions for use. The manufacturer does not accept liability for damage caused by its use for any other purpose. The user is also obliged to accept responsibility for

checking before use that the material is suitable and can be used for the intended purpose, particularly if this is not listed in the instructions for use.

Manufactured for:  
**Cosmedent, Inc.**  
401 N. Michigan Ave.  
Suite 2500  
Chicago, IL 60611-4243  
US (1) 312-644-9586  
[www.cosmedent.com](http://www.cosmedent.com)

 **DeltaMed GmbH**  
Raiffeisenstrasse 8a  
61169 Friedberg  
Germany  
[www.deltamed.de](http://www.deltamed.de)



Date 2017-11-16