

**COSMEDENT®**Creative  
Color®  
Pink Opaque™

## INSTRUCTIONS FOR USE

**www.cosmedent.com**  
**800.621.6729****GENERAL CHARACTERISTICS**

**Creative Color Pink Opaque** has been developed to mask extremely dark gray or dark brown discolorations due to tetracycline staining, the unsightly non-vital tooth and non-precious metal shine-through. Unlike other opaques, Pink Opaque has the ability to neutralize very dark colors while naturally increasing value with warmth (white opaque will not do this).

**Creative Color Pink Opaque** cures with a LED and Halogen light at wavelength range of 400 - 500 nm.

**COMPOSITION**

**Creative Color Pink Opaque** contains 1,4-Butanediol dimethacrylate, Diurethane dimethacrylate and Bis-GMA. It consists of multifunctional acrylic resins, 0.04 micron sized particles of inorganic fillers (12% by weight, 7% by volume).

**INDICATIONS FOR USE**

Mask extremely dark gray or dark brown discolorations due to tetracycline staining, the unsightly non-vital tooth and non-precious metal shine-through.

**CONTRAINDICATIONS**

**Creative Color Pink Opaque** is contraindicated if patient is known to be allergic to any of the ingredients in **Creative Color Pink Opaque** must be covered by a dental filling composite.

**SIDE EFFECTS**

May cause irritation in sensitive patients.

**NOTICE:**

During time-consuming procedures, avoid overexposure from the operatory light to protect against polymerization of the material.

**INTERACTIONS**

Do not use with eugenol containing materials, as these may impede the polymerization process.

**PLACEMENT DIRECTIONS**

- 1** Apply bonding adhesive of choice and polymerize.
- 2** On large areas, priming of the prepared surface with a thin coat of clear liquid resin to break surface tension is recommended.
- 3** Apply **Creative Color Pink Opaque** in thin layers to block out undesirable color and blend into existing tooth

structure, polymerize for 40 seconds per layer.

4

Continue with restorative resin application.

#### HYGIENE

Disinfection of the syringe can be carried out using a commercially available disinfecting solution. All instruments used should be thoroughly washed and autoclaved. Disinfect **Creative Color Pink Opaque** syringe between each patient.

#### IMPORTANT PRODUCT PRECAUTIONS

**Creative Color Pink Opaque** tips are disposable and should only be used for one patient. Cover **Creative Color Pink Opaque** syringe with a cap immediately after use.

#### WARNING

Causes skin irritation. May cause an allergic skin reaction. Causes serious eye irritation. Wear protective gloves and eye protection. In case of contact with the eyes, rinse immediately with plenty of water and seek medical attention.

#### STORAGE

- Once opened, do not refrigerate.
- Keep **Creative Color**

**Pink Opaque** at room temperature (approximately 39–75°F; 4–24°C).

- Do not use after expiration date.
- Avoid direct sunlight.
- Keep material out of reach of children.

#### PACKAGING

**Creative Color Pink Opaque** is available in syringes of 2.5g.

#### Rx IMPORTANT PRODUCT INFORMATION

Federal (U.S.) law restricts this device to sale by or on the order of a dentist.

**Note:** This product was developed for use in the dental sector and should be used in accordance with the instructions for use. The manufacturer does not accept liability for damage caused by its use for any other purpose. The user is also obliged to accept responsibility for checking before use that the material is suitable and can be used for the intended purpose, particularly if this is not listed in the instructions for use.

Manufactured for:  
**Cosmedent, Inc.**  
401 N. Michigan Ave.  
Suite 2500  
Chicago, IL 60611-4243  
US (1) 312-644-9586  
US (1) 800-621-6729  
[www.cosmedent.com](http://www.cosmedent.com)

 **DeltaMed GmbH**  
Raiffeisenstrasse 8a  
61169 Friedberg  
Germany  
[www.deltamed.de](http://www.deltamed.de)

CE 0124

Date 2018-04-23