

COSMEDENT

Brush
&
Sculpt™

LIGHT-CURE LOW VISCOSITY
RESIN COMPOSITE

INSTRUCTIONS FOR USE

www.cosmedent.com
800.621.6729

DESCRIPTION

Brush & Sculpt™ is a light-cure, low viscosity, resin composite that is approximately 36% filled. It is formulated to be used as a composite sculpting or brushing resin and will aid in overcoming the tackiness of some composites. **Brush & Sculpt™** can be used to aid in the placement and manipulation of all your composite resins. It cures with LED and Halogen light at a wavelength range of 400-500 nm.

COMPOSITION

Brush & Sculpt™ consists of multi-functional acrylic resins and 0.04 micron silica filler, 36% by weight, 24% by volume. It contains 1,4 Butandiol dimethacrylate, BIS-GMA and Diurethane dimethacrylate.

INDICATION FOR USE

Composite sculpting and brushing resin.

CONTRAINDICATIONS

Brush & Sculpt™ is contraindicated if a patient is known to be allergic to any of the ingredients in **Brush & Sculpt™**.

SIDE EFFECTS

May cause irritation in sensitive patients.

INTERACTIONS

Do not use **Brush & Sculpt™** in combination with eugenol containing materials, as these can impede the polymerization process.

DIRECTIONS FOR USE

1 Place the supplied tip on the syringe and dispense two drops into a mixing well.

2 Begin sculpting the bulk restoration by dipping a brush or placement instrument in **Brush & Sculpt™**.

3 Light cure the completed restoration for the time recommended by the composite manufacturer, typically 60 seconds.

Notice:

During time-consuming procedures, avoid overexposure from the operatory light to protect against polymerization of the material.

IMPORTANT PRODUCT PRECAUTIONS

Brush & Sculpt™ tips are disposable and should only be used for one patient. Cover

Brush & Sculpt™ syringe with a cap immediately after use.

WARNING

Causes skin irritation.
Causes serious eye irritation. May cause an allergic skin reaction.
Wear protective gloves.
In case of contact with the eyes, rinse immediately with plenty of water and seek medical attention.

STORAGE

- Keep **Brush & Sculpt™** at room temperature (approximately 39-75 °F; 4-24 °C).
- Do not use after expiration date.
- Avoid direct sunlight.
- Keep material out of reach of children.

PACKAGING

Brush & Sculpt™
1 g syringe (includes 3-22 gauge tips)

R_x IMPORTANT PRODUCT INFORMATION

Federal (U. S.) law restricts this device to sale by or on the order of a dentist.

Note:

This product was developed for use in the dental sector and should be used in accordance with the instructions for use. The manufacturer

does not accept liability for damage caused by its use for any other purpose. The user is also obliged to accept responsibility for checking before use that the material is suitable and can be used for the intended purpose, particularly if this is not listed in the instructions for use.

Manufactured For:
Cosmedent, Inc.
401 N. Michigan Ave.
Chicago, IL 60611-4243
US (1) 312-644-9586
www.cosmedent.com

 **DeltaMed GmbH**
Raiffeisenstrasse 8a
61169 Friedberg
Germany
www.deltamed.de

CE 0124

Version 2017-03-02