COSMEDENT™ www.cosmedent.com | 800.621.6729  
**COSMESEAL®** LIGHT CURED PIT & FISSURE SEALANT

## INSTRUCTIONS FOR USE

**GENERAL CHARACTERISTICS**

COSMESEAL® is a light-cured pit and fissure sealant. It is available in a filled and a clear unfilled version. COSMESEAL cures with an LED or Halogen light with a light intensity more than 500mW/cm<sup>2</sup> at a wavelength range of 400 – 500 nm.

**COMPOSITION****COSMESEAL FILLED**

COSMESEAL FILLED contains UDMA, Bis-GMA and 1,4-Butanediol dimethacrylate (63%). It is filled with 0.04 micron silicon dioxide (36% by weight and 24% by volume). Initiators and stabilizers are additional ingredients (< 1%).

**COSMESEAL CLEAR**

COSMESEAL CLEAR contains UDMA, Bis-GMA and 1,4-Butanediol dimethacrylate without filler. Initiators and stabilizers are additional ingredients (< 1%).

**INDICATIONS**

– Sealing of pits and fissures

**DISPENSING COSMESEAL**

Place the disposable tip over the syringe orifice and screw it on firmly before dispensing COSMESEAL. Throw away tip after use and immediately replace the cap on the syringe to avoid pre-polymerization.

**APPLICATION**

**For best results, it is important to have dryness of field, cleanliness of grooves to be sealed and placement of materials without interference of occlusion.**

- 1 Isolate area to be sealed with either rubber dam or absorbent cotton rolls.
- 2 Keep saliva ejector close to area of application at all times. (*Do not use high speed evacuation system.*)
- 3 Clean grooves thoroughly with a sharp explorer and prophylaxis paste before procedure. If any sign of decay remains, excavate thoroughly. Clean grooves lightly with a micro diamond bur.
- 4 Etch grooves, pits and fissures, and adjacent areas with 37% phosphoric acid. Do not etch the entire surface area. Etch for 15 seconds. Wash and dry thoroughly.
- 5 Place bonding adhesive and air thin.\*
- 6 Apply sealant by gently expressing a small amount from syringe tip. Sealant flows into grooves, but a brush with a narrow tip may be used to agitate the material gently into grooves.
- 7 Light-cure unfilled sealant for 20 seconds, and filled sealants for 40 seconds. If using high intensity light (laser or xenon), reduce polymerization time by half.
- 8 Check occlusion for interferences and remove excess. If necessary, polish. Keep completely dry throughout the procedure.

*\*It's Cosmedent's belief that the most successful way to achieve complete seal is the application of a bonding adhesive before placement of the sealant.*

**HYGIENE/DISINFECTION**

- Disinfection of the syringe can be carried out using a commercially available disinfecting solution.
- All instruments used should be thoroughly washed and autoclaved.

**STORAGE**

- Store at 39 – 75° F (4 – 24°C).
- Shelf life: see information on syringes and/or packages.
- Keep away from sunlight
- Keep material out of reach of children.
- Do not use after the expiration date.

**IMPORTANT PRODUCT PRECAUTIONS**

- COSMESEAL tips are disposable and should only be used for one patient.
- Cover COSMESEAL syringe with a cap immediately after use.
- Disinfect COSMESEAL syringes between each patient.
- Before use, place a small amount of material on a mixing pad to ensure adequate flow. If there is too much resistance, replace tip and retest.
- For better delivery of COSMESEAL, make sure that the tips are placed securely on the syringe and use slow, steady pressure to extrude the material. Do not force the syringe, as this could result in excess discharge of material.
- Use hand pieces that are oil free and air spray that is oil and moisture free.
- Care should be used in the application of etchants and sealants as these materials may irritate the mucosa.

**IMPORTANT PRODUCT INFORMATION**

**Federal (U.S.) law restricts this device to sale by or on the order of a dentist.**

**Note:** This product was developed for use in the dental sector and should be used in accordance with the instructions for use. The manufacturer does not accept liability for damage caused by its use for any other purpose. The user is also obliged to accept responsibility for checking before use that the material is suitable and can be used for the intended purpose, particularly if this is not listed in the instructions for use.

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**COMPOSITION****COSMESEAL FILLED (chargé)**

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**Cosmedent® Inc.**
**DeltaMed GmbH**