

**COSMEDENT®**GINGA-  
Guard®

Light-Curable Resin Dam

INSTRUCTIONS FOR USE

**www.cosmedent.com**  
800.621.6729**DESCRIPTION**

GINGA-Guard® is a light-cure resin used to protect the gingival tissue from irritating materials such as hydrogen peroxide. It can be used as an alternative to a rubber dam for protecting the gingival surface. GINGA-Guard cures with a LED and Halogen light at wavelength range of 400 - 500 nm.

**COMPOSITION**

GINGA-Guard consists of multi-functional acrylic resins and inorganic fillers.

**INDICATIONS**

- Protecting the gingival tissue during power bleaching.
- Protecting adjacent tooth structure or the surrounding restorations during etching with microetchants or phosphoric acid.

**CONTRAINDICATIONS**

GINGA-Guard is contra-indicated if patient is known to be allergic to any of the ingredients of GINGA-Guard. It is also contraindicated if the gingiva is inflamed.

**SIDE EFFECTS**

Localized overheating may occur from overexposure to the light-curing lamp.

**INTERACTIONS**

None.

**DISPENSING**

Place the disposable tip over the syringe orifice and screw it on firmly before dispensing the material. Throw away tip after use and immediately replace the cap on the syringe to avoid pre-polymerization.

**DIRECTIONS FOR USE**

**1**  
Apply GINGA-Guard to areas you wish to protect, i.e. gingival areas, adjacent tooth structure or surrounding restorations. Make sure the gingiva is dry before applying GINGA-Guard.

**2**  
After application, polymerize for 5 seconds on each surface.

**3**  
To remove GINGA-Guard, gently lift the material with a spoon excavator, it will come away in one piece.

**HYGIENE**

Disinfection of the syringe can be carried out using commercially available disinfecting solution. All instruments used should be thoroughly washed and autoclaved.

**IMPORTANT PRODUCT PRECAUTIONS**

GINGA-Guard tips are disposable and should only be used for one patient. Cover GINGA-Guard syringe with a cap immediately after use. Disinfect GINGA-Guard syringe between each patient.

**WARNING**

**GINGA-Guard** contains acrylic resins and may cause irritation or sensitization in some people. In case of contact with the eyes, rinse immediately with plenty of water and seek medical attention.

other purpose. The user is also obliged to accept responsibility for checking before use that the material is suitable and can be used for the intended purpose, particularly if this is not listed in the instructions for use.

**STORAGE**

- Store at 39-75° F (4 - 24°C).
- Shelf life: see information on syringes and/or packages.
- Protect GINGA-Guard syringes from heat and sunlight.
- Keep material out of reach of children.
- Do not use after the expiration date.

Manufactured for  
**Cosmedent, Inc.**  
 401 North Michigan Avenue  
 Suite 2500  
 Chicago, IL 60611-4243  
 Phone: +1-312-644-9586  
 Phone: +1 800-621-6729  
[www.cosmedent.com](http://www.cosmedent.com)

 **DeltaMed GmbH**  
 Raiffeisenstrasse 8a  
 61169 Friedberg  
 Germany  
[www.deltamed.de](http://www.deltamed.de)

**PACKAGING**

- **GINGA-Guard** 3 gram syringe - #902101
- **GINGA-Guard** Refill Tips (17 gauge) Package of 20 - #203101
- **GINGA-Guard** Refill Tips (White Plastic) Package of 25 - #203402



Version 2017-03-06

**R<sub>x</sub>**

**IMPORTANT PRODUCT INFORMATION**

Federal (U. S.) law restricts this device to sale by or on the order of a dentist.

**Note:**

This product was developed for use in the dental sector and should be used in accordance with the instructions for use. The manufacturer does not accept liability for damage caused by its use for any