## INSTRUCTIONS FOR USE

## GENERAL CHARACTERISTICS

Renamel® Flowable Microfill offers a wide shade selection, long-term color stability and excellent handling properties, with a flowable consistency that is great for quick and simple composite repairs. Renamel Flowable Microfill is the only light-cure flowable microfill available. It will provide the best esthetics and a high shine in low stress, anterior restorations. Renamel Flowable Microfill has the widest color selection and exhibits incredible long-term color stability. It is exactly color matched to all Renamel composites and the Classic VITA® Shade Guide giving you predictable results every time.

## DESCRIPTION

Renamel Flowable Microfill is a light-cured, microfilled, highly polishable dental filling material according to EN ISO 4049 (Type 1/Class 2/ Group 1).

## COMPOSITION

Renamel Flowable Microfill contains UDMA, 1,4-Butanediol dimethacrylate and Bis-GMA (47 – 48%). The fillers contain silicon dioxide and prepolymerized composite (52 – 53% by weight and 38 – 39% by volume). Initiators, stabilizers and pigments are additional ingredients (< 1%). The particle size of the fillers range from 0,04 – 0,2 micron.

## INDICATIONS FOR USE

- Seal margins
- Repair pitted areas in existing restorations
- Small occlusal and lingual restorations
- Seal pit and fissures
- Repair marginal defects
- Use in combination with Renamel Microfill in shallow Class V restorations.

## DISPENSING FLOWABLE COMPOSITE

Place the disposable tip over the syringe opening and screw it on firmly before dispensing the flowable composite. Throw away tip after use and immediately replace the cap on the syringe to avoid prepolymerization.

## PLACEMENT DIRECTIONS

Before treatment, tooth color should be assessed using a standard color guide. Keep the teeth moist when assessing the shade. For predictable results, use the same shade as your body composite. Prepare the cavity according to the general standards in composite filling therapy.

1. **Apply etching gel to entire cavity, beginning with enamel areas.** Leave the gel for 20 sec on the tooth structure (total-etch technique). Remove the etching gel and spray the tooth with water for 15 seconds. Make sure no residue remains on the tooth.
2. **Dry cavity with oil-free air.** Remoisten dentin and keep moist. Apply dentin adhesive, e. g. (Complete™ from Cosmedent) according to the Instructions for Use and cure.
3. **Apply Renamel Flowable Microfill in thin layers (max. 2 mm).** Polymerize each layer for 20 – 40 seconds. The final layer should be light cured for 60 seconds with a Halogen or LED light with a wavelength range of 400 – 500 nm.

## Seal Margins

After placing restoration and sculpting toward margins, polymerize for 40 seconds. Next, gently apply the material at the gingival margin by expressing a slight amount from the syringe along the entire margin and polymerize again, then polish. If a pit is seen at the gingival margin, roughen the area with a diamond bur, etch, wash and dry. Add bonding adhesive, then add a small amount of flowable to the defective area. Polymerize, finish and polish.

## Repair Pitted Areas in Existing Restorations

Roughen the pitted area with a broad round diamond bur, bevel edges, wash, dry and add bonding adhesive. Syringe flowable, polymerize, finish and polish.

## Occlusal Restorations

Take the smallest bur possible, such as a 330 carbide, remove decay (or use air abrasion or hard tissue laser), wash and dry the area, then etch. Add a primer, place bonding agent, polymerize and syringe flowable into cavity. Make sure the flowable extends slightly past the margin so when you polish and come back to the margin, it will not be under contoured. Remove excess material; polymerize, finish and polish.

## Shallow Class V Restorations

A small addition of Renamel Flowable Microfill prior to the application of your body microfill will make placement and sculpting much easier, due to better adhesion properties. First place your flowable microfill (same shade as body microfill) and polymerize for 40 seconds before you overlay with Renamel Microfill. Polymerize and polish.

## Temporary Restorations

Slightly reduce and abrade your completed temporary veneer or crown, then wash, clean and dry, place bonding adhesive and polymerize. Add flowable of choice to the surface, polymerize and polish.

## Sealing Open Margins

– **Newly Placed Composite**  
On a freshly placed composite restoration where the oxygen inhibited layer has not been removed, you can flow material right into the margin and it will adhere to both the dentinal margin and the restorative material, sealing margin completely. Polymerize and polish as usual.

## – Older Polished Composite

On a new or old polished restoration, the margin of the restoration to be sealed must be roughened with a medium diamond bur followed by the total-etch technique. Bonding adhesive should be

placed and polymerized, then flowable can be added to the margin, polymerized, and polished.

## FINISHING AND POLISHING

– Finishing should be done with a combination of polishing materials such as diamonds, carbide burs and aluminum oxide discs in coarse and medium grits (Mini FlexiDiscs® from Cosmedent).

- High-gloss polish can be achieved using aluminum impregnated instruments and fine superfine flexible discs (Mini FlexiDiscs® and FlexiCups®), followed by Enamelize® polishing paste applied with (Mini FlexiBuffs®) all from Cosmedent®.

## HYGIENE/DISINFECTION

Disinfection of the syringe can be carried out using a commercially available disinfecting solution. All instruments used should be thoroughly washed and autoclaved.

## STORAGE

- Once opened, do not refrigerate.
- Keep Renamel Flowable Microfill at room temperature (approximately 39 – 75 °F; 4 – 24 °C).
- Do not use after expiration date.
- Keep away from sunlight.
- Keep material out of reach of children

## IMPORTANT PRODUCT INFORMATION

Federal (U.S.) law restricts this device to sale by or on the order of a dentist.

**Note:** This product was developed for use in the dental sector and should be used in accordance with the instructions for use. The manufacturer does not accept liability for damage caused by its use for any other purpose. The user is also obliged to accept responsibility for checking before use that the material is suitable and can be used for the intended purpose, particularly if this is not listed in the instructions for use.

## CONTRAINDICATIONS

Renamel Flowable Microfill is contraindicated if patient is known to be allergic to any of the ingredients of Renamel Flowable Microfill. Renamel Flowable Microfill is contraindicated in posterior restorations in contact areas.

## SIDE EFFECTS

Can cause irritation in sensitive patients.

## INTERACTIONS

Do not use Renamel Flowable Microfill in combination with eugenol containing materials, as these can impede the polymerization process.

## NOTICE

During time-consuming procedures, avoid overexposure to the operatory light to protect against premature polymerization of the material. The light curing unit must be maintained and calibrated according to the manufacturer's specifications. Serious incidents must be reported to DeltaMed GmbH, the responsible importer and the responsible authority.

## WARNING

Renamel Flowable Microfill contains 1,4-Butanediol dimethacrylate and Urethane dimethacrylate. May cause an allergic skin reaction. Wear protective gloves. In case of contact with the eyes, rinse immediately with plenty of water and seek medical attention.

## PACKAGING

Renamel Flowable Microfill is available in 3 g syringes (includes 3-18-gauge and 3-19-gauge tips).

## SHADES

Body A1, A1.5, A2, A2.5, A3, A3.5, A4, A5, A6  
B1, B2, B3, B4,  
C1, C2, C3, C4, C5, D2, D3, D4,  
White, Light Opaque, Light Incisal, Medium Incisal, Dark Incisal

## Canada Site Licence Holder:

CLINICAL RESEARCH DENTAL  
167 Central Ave, Suite 200  
London, Ontario, Canada N6A 1M6  
Establishment License No. 666

## Australia Sponsor:

Amalgadent Dental Supplies Pty Ltd  
334 Glen Huntly Road  
Elsternwick, Victoria 3185 Australia

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**Cosmedent® Inc.**