

ADJUST™

DIRECTIONS FOR USE

Light-Cure Correction Material for Temporary Crowns & Bridges

PRODUCT DIRECTIONS:

Adjust™ is a light-cure composite correction material for repairing temporary crowns and bridges made from bis-acryl provisional materials. **Adjust** has a composition similar to **MirrorImage**, so the materials work together seamlessly to create beautiful and natural looking temporary restorations.

Indications for Adjust

- Corrections of provisionals
- Repair of temporary crowns and bridges made from bis-acryl materials.

Incompatibility with Other Materials

Do not use in combination with substances containing eugenol because eugenol can impair the hardening of the composite and cause discoloration. Also, do not allow the composite material to come into contact with, or to be stored next to products containing eugenol.

APPLICATION

Select **Adjust** in the desired VITA® shades.
Remove syringe cap and lock an angled syringe tip.
Adjust is applied directly from the syringe.

Use as a Correction Material

Apply **Adjust** directly to the parts of the provisional to be corrected. To ensure a complete cure, do not apply **Adjust** in layers exceeding 2mm.
Light-cure each layer for 40 seconds.

Use as Repair Material

a) Repair of a freshly made provisional

Preparation of the surface is not necessary.
Place **Adjust** directly from the syringe onto the fractured temporary.

b) Repair of an older temporary

Slightly roughen the fractured surface with a bur or air abrasion unit and apply a bonding adhesive (**Cosmedent's Complete™ Unfilled Resin**) according to manufacturer's instructions.

Apply **Adjust** directly from the syringe to the point of fracture. To ensure a complete cure, do not apply **Adjust** in layers exceeding 2mm.

Light cure each layer for 40 seconds.

Finishing

Contour the repair with fine diamonds, stones or burs.
Polish to a high gloss with discs or rubber points.
Interproximal finishing is accomplished by fine grit interproximal strips (**Cosmedent's FlexiStrips™**).

Basic Safety Instructions

- Do not use any resin to alter the viscosity of the correction material.
- Contact of resin pastes with skin should be avoided, especially by anyone having known resin allergies.
- Keep away from children.
- **Caution: Federal law restricts the sale of this device by or on the order of a dentist.**

Storage

- Store at 40° – 75°F (4° – 24°C).
- Do not use after expiration date.
- Close syringe immediately after use to avoid exposure to light.

ADJUST™
Light-Cure Correction Material for Temporary Crowns and Bridges
WWW.COSMEDENT.COM 800.621.6729