

ADHERE™ DIRECTIONS FOR USE

Non-Eugenol Cement for Temporary Crown and Bridge Material

PRODUCT DIRECTIONS:

Adhere™ is an automix non-eugenol temporary crown and bridge cement suitable for cementing temporary crowns or bridges as well as for trial cementation of permanent restorations. It shows excellent adhesion to temporary crown and bridge materials, but it can easily be removed from prepared teeth. **Adhere** is delivered in convenient small automix syringes making it easy to work with. Due to its non-eugenol formulation it does not influence the setting of resin-based filling and luting materials. The temporary crown and bridge cement is based on ZnO/organic acids. Especially in cases of allergies to eugenol it is strongly recommended to use this eugenol-free type of cement.

Adhere meets the requirements of the **ADA/ANSI Specification No. 30** and **DIN EN ISO 3107**.

APPLICATION

Preparing the automix syringe

Remove the cap of the automix syringe and throw it away (do not use it again.) Replace cap with a special 1:1 mixing cannula. Turn the cannula 90° until it locks in position. The material is now ready for application.

Note: Discard the first 2-3 mm of the extruded material. This must be done for each new mix. Store used syringe with fixed used mixing cannula.

Cementing

Adhere can be applied directly into the temporary crown or bridge under slight pressure from the mixing cannula. Apply a thin layer to the inside of the temporary crown or bridge material and seat cautiously but firmly over the prepared tooth structures (1 1/2 minutes working time).

Note: Do not dry prepared teeth completely. A slightly moist surface facilitates the removal of your temporary crown or bridge material later on.

Adhere sets in the mouth within 2-3 minutes. Once **Adhere** has set, remove any excess.

Basic Safety Instructions

- Do not use **Adhere** in cases where known allergies to the components exist.
- Do not use **Adhere** if in some cases individual contact allergies may exist.
- In case of contact with skin or eyes rinse immediately with plenty of water and consult a physician if necessary.
- If swallowed seek medical advice.
- Keep away from children.
- **Caution: Federal law restricts the sale of this device by or on the order of a dentist.**

Storage

- Store at 47° – 75°F (8° - 24°C).
- **Adhere should not be stored at temperatures below 47° F (8° C).**
- Do not use after expiration date.

ADHERE™
Non-Eugenol Cement for Temporary Crown and Bridge Material
WWW.COSMEDENT.COM 800.621.6729