EN

COSMEDENT™ www.cosmedent.com | 800.621.6729

Brush & Sculpt™ | LIGHT-CURE LOW VISCOSITY RESIN COMPOSITE

INSTRUCTIONS FOR USE

DESCRIPTION

Brush & Sculpt™ is a light-cure, low viscosity, resin composite that is approximately 36% filled. It is formulated to be used as a composite sculpting or brushing resin and will aid in overcoming the tackiness of some composites. **Brush & Sculpt** can be used to aid in the placement and manipulation of all your composite resins. It cures with an LED or Halogen light at a wavelength range of 400-500 nm.

COMPOSITION

The monomer matrix of **Brush & Sculpt** contains UDMA, Bis-GMA and 1,4-Butanediol dimethacrylate (63%). It is filled with 0.04 micron sized silicon dioxide (36% by weight and 24% by volume). Initiators and stabilizers are additional ingredients (< 1%).

INDICATIONS FOR USE

Composite sculpting and brushing resin.

DIRECTIONS FOR USE

- Place the supplied tip on the syringe and dispense two drops into a mixing well.
- Begin sculpting the bulk restoration by dipping a brush or placement instrument in **Brush & Sculpt**.
- Light cure the completed restoration for the time recommended by the composite manufacturer, typically 60 seconds.

HYGIENE / DISINFECTION

- Disinfection of the syringe can be carried out using a commercially available disinfecting solution.
- All instruments used should be thoroughly washed and autoclaved.

IMPORTANT PRODUCT PRECAUTIONS

Brush & Sculpt tips are disposable and should only be used for one patient. Cover **Brush & Sculpt** syringe with a cap immediately after use.

STORAGE

- Keep **Brush & Sculpt** at room temperature (approximately 39-75 °F; 4-24 °C).
- Do not use after expiration date.
- Keep away from sunlight
- Keep material out of reach of children.

Rx IMPORTANT PRODUCT INFORMATION

Federal (U.S.) law restricts this device to sale by or on the order of a dentist.

Note: This product was developed for use in the dental sector and should be used in accordance with the instructions for use. The manufacturer does not accept liability for damage caused by its use for any other purpose. The user is also obliged to accept responsibility for checking before use that the material is suitable and can be used for the intended purpose, particularly if this is not listed in the instructions for use.

DISPOSAL INSTRUCTIONS

Dispose of contents/ container in accordance with local regulation.

CONTRAINDICATIONS

Brush & Sculpt is contraindicated if a patient is known to be allergic to any of the ingredients in **Brush & Sculpt**.

SIDE EFFECTS

May cause irritation in sensitive patients.

INTERACTIONS

Do not use **Brush & Sculpt** in combination with eugenol containing materials, as these can impede the polymerization process.

NOTICE

During time-consuming procedures, avoid overexposure from the operatory light to protect against premature polymerization of the material. The light curing unit must be maintained and calibrated

according to the manufacturer's specifications. Serious incidents must be reported to DeltaMed GmbH, the responsible importer and the responsible authority.

WARNING

Brush & Sculpt contains 1,4-Butandiol dimethacrylate and Diurethane dimethacrylate.

Causes skin irritation. Causes serious eye irritation. May cause an allergic skin reaction. Wear protective gloves. In case of contact with the eyes, rinse immediately with plenty of water and seek medical attention.

PACKAGING

Brush & Sculpt is available in 1 g syringe (includes 3-22 gauge tips).

Canada Site Licence Holder:
CLINICAL RESEARCH DENTAL
167 Central Ave, Suite 200
London, Ontario, Canada N6A 1M6
Establishment License No. 666

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Cosmedent® Inc.