

COSMEDENT[®] INSURE[®] Color Guide



Insure Cement	Shade Range	To Lighten	To Darken	To Opaque*
CLEAR	B1, A1, B2, D2, Oxygen™ Try-in matches shade-Veneer to be same shade as tooth	Add a small amount of White Opaque (WO) resin cement (e.g., 1 part WO to 10 parts of CLEAR)	Add some YR-LT or YR-UNIV if in A or B shades. Add some G-LT if in C or D shades	B1: add WO to CLEAR A1: add PO to CLEAR B2: add YO to CLEAR D2: add PO to CLEAR
YR-LT (Yellow Red Light)	A2, A3	Use CLEAR	Use YR-UNIV	Use PO or mix YR-LT and PO
YR-UNIV (Yellow Red Universal)	B3, A3.5, B4, A4	Add CLEAR		Mix YR-UNIV and YO or use YO for B shades Mix YR-UNIV and PO for A shades
G-LT (Grey Light)	D3, C3, C4	Use CLEAR		Mix G-LT and PO or use PO
WO (White Opaque)	Lighter than shade guide teeth "Chicklet look"	Add White Color Modifier	Use CLEAR or mix WO and CLEAR	Add White Color Modifier
YO (Yellow Opaque)	Lighter B shades	Add White Color Modifier	Use CLEAR or mix YO and CLEAR	Add White Color Modifier
PO (Pink Opaque)	Lighter A, C and D shades	Add White Color Modifier	Use CLEAR or mix PO and CLEAR	Add White Color Modifier



(800) 621-6729 • www.cosmedent.com

©Copyright 2022 Cosmedent, Inc. All rights reserved.