EN

COSMEDENT<sup>®</sup>

www.cosmedent.com | 800.621.6729

Insure<sup>®</sup> Silanator

## INSTRUCTIONS FOR USE

## DESCRIPTION

**Insure<sup>®</sup> Silanator** is a single component adhesive coupling agent that creates a permanent bond between silica-based ceramics and resin based restorative materials.

## COMPOSITION

**Insure Silanator** consists of a water / ethanol solution of a modified hydrolyzed trimethoxysilane.

## INDICATIONS FOR USE

(Areas of application)

- Adhesive coupling agent for silica based ceramics and methacrylate-based composite restorative materials and cements.

## APPLICATION

## Pretreatment of silica ceramics:

Etch the intaglio surface of the ceramic restoration with a hydrofluoric acid etching gel according to the manufacturer's instructions. Rinse thoroughly with water and dry with oil-free air.

- ① Apply **Insure Silanator** to the etched surface with a brush, allow to remain in place for 30 seconds followed by gently drying with the air syringe until the surface is completely dry.
- ② The silanated surfaces should be protected from contamination (e.g. blood or saliva).
- ③ Apply a universal adhesive such as **Cosmedent Complete<sup>™</sup>** followed by application of a resin cement such as **Insure<sup>®</sup>**.

## HYGIENE/DISINFECTION

- All instruments used should be thoroughly washed and autoclaved.

## STORAGE

- Store at 39 – 75°F (4 – 24°C).
- Shelf life: see information on bottle.
- Keep away from sunlight.
- Keep material out of reach of children.
- Do not use after the expiration date.

**Rx** IMPORTANT  
PRODUCT  
INFORMATION

Federal (U.S.) law restricts this device to sale by or on the order of a dentist.

**Note:** This product was developed for use in the dental sector and should be used in accordance with the instructions for use. The manufacturer does not accept liability for damage caused by its use for any other purpose. The user is also obliged to accept responsibility for checking before use that the material is suitable and can be used for the intended purpose, particularly if this is not listed in the instructions for use.

## DISPOSAL INSTRUCTIONS

Dispose of contents/ container in accordance with local regulation.

## CONTRAINDICATIONS

Placement of **Insure Silanator** is contraindicated

- If a patient is known to be allergic to any of the ingredients.
- On alumina or zirconia restorations. Contact porcelain manufacturer if in doubt.

## SIDE EFFECTS

May cause irritation in sensitive patients.

## INTERACTIONS

Contamination of the silanated surface with saliva, blood or water must be avoided.

## NOTICE

Serious incidents must be reported to DeltaMed GmbH, the responsible importer and the responsible authority.

## WARNING

**Insure Silanator** contains ethanol. Flammable liquid and vapor. Keep away from ignition sources. Wear protective gloves.

## PACKAGING

**Insure Silanator** is available in 10 ml bottles.

Canada Site Licence Holder:  
CLINICAL RESEARCH DENTAL  
167 Central Ave, Suite 200  
London, Ontario, Canada N6A 1M6  
Establishment License No. 666

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Schwerwiegende Vorfälle sind der DeltaMed GmbH, dem zuständigen Importeur und der zuständigen Behörde zu melden.