da vinci Formula 1™ Resin Cement | DA VINCI RESIN CEMENTATION SYSTEM

INSTRUCTIONS FOR USE

DESCRIPTION

da Vinci Formula 1™ Resin Cement is a light cured, microhybrid filled, radiopaque, dental resin cement according to EN ISO 4049 (Type2/Class2/Group1). It is recommended to be used for ceramic laminate veneers. **da Vinci Formula 1 Resin Cement** cures with an LED or Halogen light with a light intensity more than 500mW/cm² at wavelength range of 400 – 500 nm.

COMPOSITION

da Vinci Formula 1 Resin Cement contains Bis-GMA, UDMA and 1,4-Butanediol dimethacrylate (30%). The fillers contain barium glass and silicon dioxide (69% by weight and 46% by volume). Initiators and stabilizers are additional ingredients (< 1%). The particle size of the fillers range from 0.04 – 3 micron.

INDICATIONS FOR USE

(Areas of application)
– Cementation of ceramic veneers

DISPENSING DA VINCI FORMULA 1 RESIN CEMENT

Place the disposable tip over the syringe opening and place it on firmly before dispensing the **da Vinci Formula 1 Resin Cement**. This will provide controlled placement into the veneer shell without excess resin cement extruding from the syringe causing material waste. Throw away tip after use and immediately replace the cap on the **da Vinci Formula 1 Resin Cement** syringe to avoid pre-polymerization.

EVALUATING FOR MARGINAL FIT

Prior to cementation of ceramic laminate veneers, it is advisable to evaluate each ceramic laminate veneer for precise marginal fit and interproximal adaptation to adjacent ceramic laminate veneers or natural tooth structure. Any aberrations should be identified, and appropriate measures set up for their elimination.

SELECTING SHADE

We recommend that you use **da Vinci Formula 1 Try-In Gels** to select the correct shade of your **da Vinci Formula 1 Resin Cement**. **da Vinci Formula 1 Try-In Gels** are especially effective for feldspathic porcelain veneers or any minimal or no-prep veneers. These non-light activated pastes accurately match the polymerized color of **da Vinci Formula 1 Resin Cement** and wash away quickly with water, making shade selection predictable and fast.

PLACEMENT DIRECTIONS

① Once the ceramic laminate veneers have been tried in and approved for marginal fit, proximal adaptation, and resin cement color, the following cementation procedure is undertaken:

② Isolate the teeth to be veneered with a rubber dam or cotton rolls and an effective evacuation system that will sustain a dry uncontaminated field throughout the duration of the cementation procedure.

③ Place retraction cord, if needed, along the cervical margins of all teeth to be veneered to permit clear visualization of the cervical finish lines and counteract the potential for soft tissue contamination of any kind.

④ Floss each contact to remove any residual provisional veneer resin or resin cement. Relax but do not open the contacts with controlled passage of narrow, SuperFine, metal backed **FlexiStrips™** (Cosmedent). Remove any residual provisional veneer cement on the facial surface with a fine diamond bur.

⑤ Clean all tooth surfaces to be veneered by initially pumicing all accessible tooth surfaces with pumice. Rinse away all pumice residue and microetch any existing resin restorations to which the veneers will be cemented. Rinse all tooth surfaces to be cemented thoroughly with water and air dry.

If the laboratory has not acid etched and silanated the restorations or if this surface is altered by the dentist, then these steps must be performed prior to resin bonding. If the laboratory has etched the porcelain, skip to step 5c.

– a) Etch the intaglio of the veneer for 20 seconds with the 5% Porcelain etchant (e.max™, Ivoclar Vivadent) or 3 minutes with 9.5% Hydrofluoric acid if Feldspathic porcelain.

b) Rinse thoroughly with water and air dry with oil free air to visualize a frosted, etched appearance.

c) Apply a silanizing agent such as **Insure® Silanator** (Cosmedent) for 30 seconds to all etched inner surfaces of the veneers and gently air dry. Place the ceramic laminate veneers in sequential order for cementation in pairs, starting with the central incisors, or in quartets, starting with the incisors.

– Place a thin metal strip interproximally on the distal proximal surface of the last prepared tooth to be cemented in this grouping (either a pair of veneers or 4 consecutive veneers starting at the midline), to prevent etching an adjacent tooth not in this cementation grouping. Etch each tooth surface to be veneered with 35%–37% phosphoric acid for approximately 20 seconds and then rinse off all acid with a gentle water spray for 5 seconds and carefully dry the teeth with a clean air spray to a damp, but not desiccated state.

⑥ Place dental adhesive of choice, such as Cosmedent's **Complete™**, according to the manufacturer's instructions. **Do not light-cure**. Lower the dental headlight to chin level to prevent premature polymerization. Remove all interproximal metal strips.

⑦ Load the veneers to be placed with the selected **da Vinci Formula 1 Resin Cement**. Be sure to place the disposable tip over the syringe opening and place it on firmly before dispensing the **da Vinci Formula 1 Resin Cement**. This will provide controlled placement into the veneer shell without excess resin cement extruding from the syringe causing material waste. Be sure to load each veneer fully to the margin because underloading can cause voids.

⑧ Seat the veneer(s) from the incisofacial or facial, using the incisal porcelain as the initial seating guide. Finalize the seating with gentle finger pressure, as the excess **da Vinci Formula 1 Resin Cement** gradually extrudes along the margins. Remove resin cement excess with an **IPC Composite Placement Instrument** (Cosmedent) at the cervical and incisal. Remove all but a narrow string of excess resin cement (0.2 mm wide) at the margin to offset suckback porosity or "crevicing" due to polymerization shrinkage. **Spot-cure** the veneer with a 2 or 3 mm curing tip on the facial surface away from the margins for **10 seconds** to secure it in place. Further refine resin cement removal along the margins as needed with dental floss before light-curing the rest of the veneer. Place glycerin such as **Oxygene™** (Cosmedent) along the margins. **Light-cure** (preferable with 2 lights at once) the veneer for at least **60 seconds facially** and **60 seconds incisolingually** with a wide diameter curing light.

⑨ Remove ceramic resin cement flash along the margins with metal-backed **FlexiStrips** (Cosmedent), Bard Parker 12B scalpel blades and **Mini FlexiDiscs** (Cosmedent). If ceramic "ledging" requires adjustment, use microfine diamonds with a copious water spray, All Ceramic Polishers (Cosmedent) and polishing paste to renew the glaze.

⑩ Check the occlusion in centric and eccentric movements with fine articulating paper. Adjust the occlusion where necessary. Repolish the adjusted surface as suggested in #9.

HYGIENE/DISINFECTION

– Disinfection of the syringe can be carried out using a commercially available disinfecting solution.
– All instruments used should be thoroughly washed and autoclaved.

STORAGE

– Store at 39 – 75°F (4 – 24°C).
– Shelf life: see information on syringes and/or packages.
– Keep away from sunlight.
– Keep out of reach of children.
– Do not use after the expiration date.

Rx IMPORTANT PRODUCT INFORMATION

Federal (U.S.) law restricts this device to sale by or on the order of a dentist.

Note: This product was developed for use in the dental sector and should be used in accordance with the instructions for use. The manufacturer does not accept liability for dam- age caused by its use for any other purpose. The user is also obliged to accept responsibility for checking before use that the material is suitable and can be used for the intended purpose, particularly if this is not listed in the instructions for use.

DISPOSAL INSTRUCTIONS

Dispose of contents/ container in accordance with local regulation.

CONTRAINDICATIONS

Placement of **da Vinci Formula 1 Resin Cement** is contraindicated if a patient is known to be allergic to any of the ingredients of **da Vinci Formula 1 Resin Cement**.

SIDE EFFECTS

May cause irritation in sensitive patients.

INTERACTIONS

Do not use in combination with eugenol containing materials as these may impede the polymerization process.

NOTICE

During time-consuming procedures, avoid overexposure from the operatory light to protect against premature polymerization of the material. The light curing unit must be maintained and calibrated according to the manufacturer's specifications. Serious incidents must be reported to DeltaMed GmbH, the responsible importer and the responsible authority.

WARNING

da Vinci Formula 1 Resin Cement contains 1,4-Butanediol dimethacrylate and Urethane dimethacrylate. Causes skin irritation. May cause an allergic skin reaction. Causes serious eye irritation. Wear protective gloves. In case of contact with the eyes, rinse with plenty of water and seek medical attention.

PACKAGING

da Vinci Formula 1 Resin Cement is available in 3 g syringes (includes 3 nozzle tips).

SHADES

da Vinci Formula 1 Resin Cement 3g is available in the following shades: Clear, Bright, Brighter, Brightest, Ludicrous Bright.

Canada Site Licence Holder:
CLINICAL RESEARCH DENTAL
167 Central Ave, Suite 200
London, Ontario, Canada N6A 1M6
Establishment License No. 666

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