EN

COSMEDENT™ www.cosmedent.com | 800.621.6729

## da vinci Formula 1™ Try-In Gels

## INSTRUCTIONS FOR USE

**DESCRIPTION**

da vinci Formula 1™ Try-In Gels are water-soluble try-in gels take the guesswork and messiness out of porcelain and indirect resin veneer color evaluation. da vinci Formula 1 Try-In Gels are exactly color-matched to polymerize da vinci Formula 1™ Resin Cements.

Veneers are loaded with da vinci Formula 1 Try-In Gel, and placed over all prepared teeth, permitting complete visualization of the veneer by the patient.

da vinci Formula 1 Try-In Gels are easily removed from veneers and tooth structure with a water spray.

**COMPOSITION**

da vinci Formula 1 Try-In Gel consists of propylene glycol, glycerin, water, acrylic polymer and pigments. It is pH neutral.

**INDICATIONS FOR USE**

(Areas of application)  
– Water soluble try-in veneer cement system.

**DISPENSING DA VINCI FORMULA 1 TRY-IN GEL**

Place the disposable tip over the syringe opening and place it on firmly before dispensing the da vinci Formula 1 Try-In Gel. This will provide controlled placement into the veneer shell without excess Try-In Gel extruding from the syringe causing material waste. Throw away tip after use and replace the cap on the Try-In Gel syringe.

**APPLICATION**

Before treatment, tooth shade should be taken using a standard shade guide. Keep teeth moist when taking the shade.

- 1 Select the appropriate da vinci Formula 1 Try-In Gel shade or blend for trial evaluation.
- 2 Fully load the previously etched internal surface of each veneer with try-in gel.
- 3 Clean and dry all teeth receiving the veneers, but do not etch the teeth or condition the dentin.
- 4 Place each veneer on the recipient tooth and seat completely with light pressure. Wipe away any excess try-in gel from the margins.
- 5 Place the patient in an upright position and evaluate color before a large mirror under multiple lighting conditions, e.g. daylight, office lighting, etc.
- 6 If the color is favorable, select the corresponding da vinci Formula 1 Resin Cement or resin cement mix, e.g. Clear da vinci Formula 1 Try-In Gel would require Clear da vinci Formula 1 Resin Cement.
- 7 Remove the veneers from each tooth. With a vigorous water spray, rinse the da vinci Formula 1 Try-In Gel from the veneers and the teeth.

**HYGIENE/DISINFECTION**

– Disinfection of the syringe can be carried out using a commercially available disinfecting solution.  
– All instruments used should be thoroughly washed and autoclaved.

**STORAGE**

– Store at 39 – 75°F (4 – 24°C).  
– Shelf life: see information on syringes and / or packages.  
– Keep out of reach of children.  
– Keep away from sunlight  
– Do not use after the expiration date

**IMPORTANT PRODUCT INFORMATION**

Federal (U.S.) law restricts this device to sale by or on the order of a dentist.

**Note:** This product was developed for use in the dental sector and should be used in accordance with the instructions for use. The manufacturer does not accept liability for damage caused by its use for any other purpose. The user is also obliged to accept responsibility for checking before use that the material is suitable and can be used for the intended purpose, particularly if this is not listed in the instructions for use.

**DISPOSAL INSTRUCTIONS**

Dispose of contents/ container in accordance with local regulation.

**CONTRAINDICATIONS**

Placement of da vinci Formula 1 Try-In Gel is contraindicated if a patient is known to be allergic to any of the ingredients of da vinci Formula 1 Try-In Gel.

**SIDE EFFECTS**

May cause irritation

**INTERACTIONS**

None

**NOTICE**

Serious incidents must be reported to DeltaMed GmbH, the responsible importer and the responsible authority

**WARNING**

da vinci Formula 1 Try-In Gels contain glycerin and propylene glycol and may cause irritation in sensitive persons. In case of contact with the eyes, rinse immediately with plenty of water and seek medical attention if irritation persists.

**PACKAGING**

da vinci Formula 1 Try-In Gels are available in 2.5 g syringes (includes 3-19 gauge tips).

**SHADES**

da vinci Formula 1 Try-In Gels are available in the following shades, color-matched to the da vinci Formula 1 Resin Cement shades: Clear, Bright, Brighter, Brightest, Ludicrous Bright.

Canada Site Licence Holder:  
CLINICAL RESEARCH DENTAL  
167 Central Ave, Suite 200  
London, Ontario, Canada N6A 1M6  
Establishment License No. 666

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Schwerwiegende Vorfälle sind der DeltaMed GmbH, dem zuständigen Importeur und der zuständigen Behörde zu melden.
Cosmedent® Inc.