

# CLASS IV RESTORATION



**COSMEDENT®** *Spreading Smiles*

**COSMEDENT**

401 N. Michigan Ave. Suite 2500, Chicago, IL 60611

**Smiling Patients, Smiling Dentists.**

Helping you succeed with quality products and dedicated support

# CLASS IV RESTORATION


Dr. Bud Mopper



Class VI



Create a 2 millimeter long bevel on the facial.

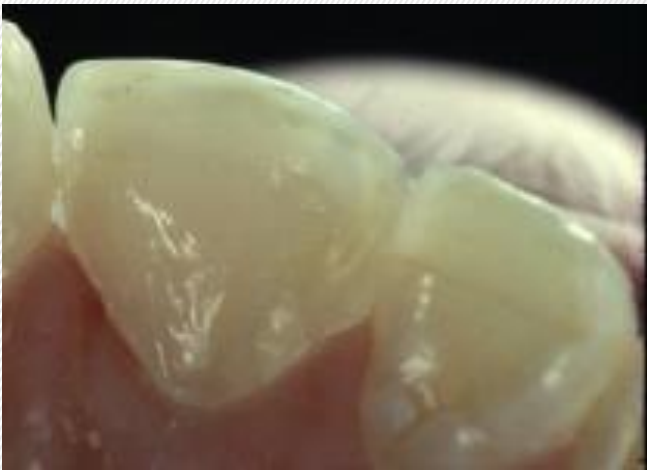

Create a butt joint on the lingual.



Place Renamel Nano to extended to correct the incisal length



Renamel Nano is applied leaving room for opaque and microfill.



Notice the lower value at the Incisal edge.



Creative Color Opaque of the same shade as the Renamel Nano is used to increase the value before placement of Renamel Microfill.



After placement of Renamel Microfill always check from the incisal view to ensure proper contour. Always keep the proximal surfaces clean and overbulk on the facial prior to finishing and polishing.



Final Restoration after finishing and polishing.

Note: Color blending, perfect color match, and incisal translucency



## One Shade Wonder Bundle



**SAVE  
20%**

800.621.6729 [www.cosmedent.com](http://www.cosmedent.com)