EN

COSMEDENT™ www.cosmedent.com | 800.621.6729  
Creative Color® Pink Opaque

## INSTRUCTIONS FOR USE

## GENERAL CHARACTERISTICS

**Creative Color® Pink Opaque** has been developed to mask extremely dark gray or dark brown discolorations due to tetracycline staining, the unsightly non-vital tooth and non-precious metal shine-through. Unlike other opaques, **Pink Opaque** has the ability to neutralize very dark colors while naturally increasing value with warmth (white opaque will not do this).

**Creative Color Pink Opaque** cures with an LED or Halogen light at wavelength range of 400 - 500 nm.

## COMPOSITION

**Creative Color Pink Opaque** contain UDMA, Bis-GMA and 1,4-Butanediol dimethacrylate (84%). It is filled with 0.04 micron sized silicon dioxide (14% by weight and 8% by volume). Initiators, stabilizers and pigments are additional ingredients (<2%).

## INDICATIONS FOR USE

Mask extremely dark gray or dark brown discolorations due to tetracycline staining, the unsightly non-vital tooth and non-precious metal shine-through.

## PLACEMENT DIRECTIONS

① Apply bonding adhesive of choice and polymerize.

② On large areas, priming of the prepared surface with a thin coat of **Clear Liquid Resin** to break surface tension is recommended.

③ Apply **Creative Color Pink Opaque** in thin layers to block out undesirable color and blend into existing tooth structure, polymerize for 40 seconds per layer.

④ Continue with restorative resin application.

## HYGIENE/DISINFECTION

- Disinfection of the syringe can be carried out using a commercially available disinfecting solution.
- All instruments used should be thoroughly washed and autoclaved.
- Disinfect **Creative Color Pink Opaque** syringe between each patient.

## IMPORTANT PRODUCT PRECAUTIONS

- **Creative Color Pink Opaque** tips are disposable and should only be used for one patient.
- Cover **Creative Color Pink Opaque** syringe with a cap immediately after use.

## STORAGE

- Once opened, do not refrigerate.
- Keep **Creative Color Pink Opaque** at room temperature (approximately 39-75°F; 4-24°C).
- Do not use after expiration date.
- Keep away from sunlight.
- Keep material out of reach of children.

**Rx** IMPORTANT  
PRODUCT  
INFORMATION

Federal (U.S.) law restricts this device to sale by or on the order of a dentist.

**Note:** This product was developed for use in the dental sector and should be used in accordance with the instructions for use. The manufacturer does not accept liability for damage caused by its use for any other purpose. The user is also obliged to accept responsibility for checking before use that the material is suitable and can be used for the intended purpose, particularly if this is not listed in the instructions for use.

## DISPOSAL INSTRUCTIONS

Dispose of contents/ container in accordance with local regulation

## CONTRAINDICATIONS

**Creative Color Pink Opaque** is contraindicated if patient is known to be allergic to any of the ingredients in **Creative Color Pink Opaque**. **Creative Color Pink Opaque** must be covered by a dental filling composite.

## SIDE EFFECTS

May cause irritation in sensitive patients.

## INTERACTIONS

Do not use with eugenol containing materials, as these may impede the polymerization process.

## NOTICE

During time-consuming procedures, avoid overexposure from the operatory light to protect against premature polymerization of the material. The light curing unit must be maintained and calibrated according to the manufacturer's specifications. Serious incidents must be reported to DeltaMed GmbH, the responsible importer and the responsible authority.

## WARNING

**Creative Color Pink Opaque** contains 1,4-Butanediol dimethacrylate and Urethane dimethacrylate. Causes skin irritation. May cause an allergic skin reaction. Causes serious eye irritation. Wear protective gloves. In case of contact with the eyes, rinse with plenty of water and seek medical attention.

## PACKAGING

**Creative Color Pink Opaque** is available in syringes of 2.5g.

Canada Site Licence Holder:  
CLINICAL RESEARCH DENTAL  
167 Central Ave, Suite 200  
London, Ontario, Canada N6A 1M6  
Establishment License No. 666

Australia Sponsor:  
ERSKINE ORAL CARE  
5-9 Kylie St,  
Macksville NSW 2447  
Australia

Australia Sponsor:  
Amalgadent Dental Supplies Pty Ltd  
334 Glen Huntly Road  
Elsternwick, Victoria 3185 Australia

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**Cosmedent® Inc.**