EN

COSMEDENT®

www.cosmedent.com | 800.621.6729

Creative Color®

Clear Liquid Resin

INSTRUCTIONS FOR USE

DESCRIPTION

Creative Color® Clear Liquid Resin is an unfilled acrylic resin. It works in conjunction with **Creative Color Opaquers** and **Tints**.

Creative Color Clear Liquid Resin cures with an LED or Halogen light at wavelength range of 400 – 500 nm.

COMPOSITION

Creative Color Clear Liquid Resin consists of UDMA, Bis-GMA and 1,4-Butanediol dimethacrylate. Initiators and stabilizers are additional ingredients (< 1%).

INDICATIONS FOR USE

(Areas of application)

- To modify the consistency of tints and opaquers.
- Break surface tension to exposed tooth surface prior to the direct application of opaquers.

APPLICATION

If using to break surface tension:

- Apply adhesive of choice, such as **Cosmedent Complete™**, according to manufacturer's instructions. Air thin and polymerize.
- Apply thin coat of clear liquid resin and polymerize for 10 seconds.
- Apply your opaquer.

If using to modify consistency

- Mix desired amount of **Creative Color Clear Liquid Resin** with your tint or opaquer until desired consistency is achieved.

STORAGE

- Store at 39 – 75°F (4 – 24°C).
- Shelf life: see information on syringes and/or packages.
- Keep away from sunlight.
- Keep out of reach of children.
- Do not use after the expiration date

IMPORTANT PRODUCT INFORMATION

Federal (U.S.) law restricts this device to sale by or on the order of a dentist.

Note: This product was developed for use in the dental sector and should be used in accordance with the instructions for use. The manufacturer does not accept liability for damage caused by its use for any other purpose. The user is also obliged to accept responsibility for checking before use that the material is suitable and can be used for the intended purpose, particularly if this is not listed in the instructions for use.

DISPOSAL INSTRUCTIONS

Dispose of contents/ container in accordance with local regulation.

CONTRAINDICATIONS

Placement of **Creative Color Clear Liquid Resin** is contraindicated if a patient is known to be allergic to any of the ingredients.

SIDE EFFECTS

Can cause irritation in sensitive patients

INTERACTIONS

Do not use in combination with eugenol containing materials as these may impede the polymerization process.

NOTICE

During time-consuming procedures, avoid overexposure from the operatory light to protect against premature polymerization of the material. The light curing unit must be maintained and calibrated according to the manufacturer's specifications. Serious incidents must be reported to DeltaMed GmbH, the responsible importer and the responsible authority.

WARNING

Creative Color Clear Liquid Resin contains 1,4-Butanediol dimethacrylate and Urethane dimethacrylate. Causes skin irritation. May cause an allergic skin reaction. Causes serious eye irritation. Wear protective gloves. In case of contact with the eyes, rinse with plenty of water and seek medical attention.

PACKAGING

Creative Color Clear Liquid Resin is available in a bottle of 3ml.

Australia Sponsor:
ERSKINE ORAL CARE
5-9 Kylie St,
Macksville NSW 2447
Australia

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Cosmedent® Inc.