EN

COSMEDENT® www.cosmedent.com | 800.621.6729

Creative Color® Opaquers

INSTRUCTIONS FOR USE

CREATIVE COLOR SYSTEM OVERVIEW

Creative Color® Tints and Opaquers have been the industry standard for almost 30 years, and no composite system is complete without them. The combination of tints and opaquers are designed to augment the esthetic result one can achieve with composite bonding. Opaquers and tints are key ingredients to creating invisible restorations. Opaquers enable you to block-out unwanted color and translucent shine-through, while tints help you to bring the desired color back into the tooth and mimic translucence. The use of both is where your creativity as a dentist can really shine. Creative Color is a system of light-cured, brushable tints and opaquers for use in conjunction with the Renamel Restorative System™.

GENERAL CHARACTERISTICS

Creative Color Opaquers help create invisibility by blocking undesirable under-color and unwanted translucent shine-through. Cosmedent has calibrated both the color and the brightness of Creative Color Opaquers to the Classic VITA® Shade Guide and all Renamel composites so they blend invisibly into your restoration while blocking translucent shine-through at the same time.

Creative Color Opaquers are compatible with all composite materials. The viscosity is such that the material will flow with slight manipulation of the material with a brush. Creative Color Opaquers are microfill based, allowing light to shine through them for a more natural looking restoration. The use of Cosmedent brushes is recommended.

You are in control of this material, not the other way around.

- Easy to manipulate and control.

A viscosity that flows only where you want it; a liquid that moves only when you touch it.

- Properly pigmented to cover without great bulk.

Opaquers should be applied in ultrathin layers.

- Thoroughly Polymerizable

Each layer should be cured for 40 seconds.

- Color Coordinated to the Classic VITA Shade Guide

You can seamlessly blend the corresponding opaque shade into your composite for invisible results.

Creative Color Opaquers cure with an LED or Halogen light at wavelength range of 400 - 500 nm.

COMPOSITION

Creative Color Opaquers contain UDMA, Bis-GMA and 1,4-Butanediol dimethacrylate (84%). They are filled with 0.04 micron sized silicon dioxide (14% by weight and 8% by volume). Initiators, stabilizers and pigments are additional ingredients (up to 2%).

INDICATIONS FOR USE

- To increase value or brightness
- Create opacity to block-out translucent shine-through
- Block-out undesirable color of tooth structure while color-blending naturally into your restoration
- Create surface characterization such as hypoplasia and check lines
- Color change

DISPENSING MATERIAL

Dispense the material onto a mixing pad. Apply with a brush directly to the tooth surface. Replace the cap on the Creative Color syringe to avoid pre-polymerization.

PLACEMENT DIRECTIONS

- ① Opaquers can be applied directly to the tooth surface after preparation and placement of adhesive of choice, or over your composite prior to microfill overlay.
- ② Select the appropriate opaquer in the same shade as your final composite layer. If you wish to change the viscosity of the opaquer, gently stir with plastic instrument or mix with Creative Color® Clear Liquid Resin™.
- ③ Place with a Cosmedent brush until the area is completely covered with a very thin layer of opaquer. Use the brush gingerly for a smooth, even spread. On broad surfaces, the #3 brush (from Cosmedent) is desirable. To spot opaque, the #1 brush (from Cosmedent) is desirable.
- ④ **Polymerize for 40 seconds per surface.**
- ⑤ Use a second and third coat if there is still undercolor shining through. **Polymerize each layer separately.**
- ⑥ Cover with Renamel® Microfill or the composite of choice. **Polymerize for 60 seconds.** Contour and polish.

Remember, add opaque slowly and with care to prevent over-opacity. It may be necessary to add in sections on large restorations.

For more detailed polishing techniques, please visit Cosmedent's Learning Center at Cosmedent.com/Learning_Center or call a Cosmedent Customer Care Representative at 800.621.6729.

HYGIENE/DISINFECTION

- Disinfection of the syringe can be carried out using a commercially available disinfecting solution.
- All instruments used should be thoroughly washed and autoclaved.
- Disinfect Creative Color Opaquers syringe between each patient.

IMPORTANT PRODUCT PRECAUTIONS

- Creative Color Opaquers tips are disposable and should only be used for one patient.
- Seal Creative Color Opaquers syringe with a cap immediately after use.

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DISPOSAL INSTRUCTIONS

Dispose of contents/container in accordance with local regulation.

CONTRAINDICATIONS

Creative Color Opaquers are contraindicated if patient is known to be allergic to any of the ingredients. Creative Color Opaquers must be covered by a dental filling composite.

SIDE EFFECTS

Can cause irritation in sensitive patients.

INTERACTIONS

Do not use with eugenol containing materials, as these may impede the polymerization process.

NOTICE

During time-consuming procedures, avoid overexposure from the operatory light to protect against premature polymerization of the material. The light curing unit must be maintained and calibrated according to the manufacturer's specifications. Serious incidents must be reported to DeltaMed GmbH, the responsible importer and the responsible authority.

WARNING

Creative Color Opaquer contains 1,4-Butanediol dimethacrylate and Urethane dimethacrylate. Causes skin irritation. May cause an allergic skin reaction. Causes serious eye irritation. Wear protective gloves. In case of contact with the eyes, rinse with plenty of water and seek medical attention.

PACKAGING

Creative Color Opaquers are available in syringes of 2.5g.

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