Insure® Lite Automix | Universal Cementation System

INSTRUCTIONS FOR USE

GENERAL CHARACTERISTICS

Insure® Lite Automix is a dual-cure, automix, hybrid filled, radiopaque, resin cement according to EN ISO 4049 (Typ2/Class3). The automix syringe is a dual cartridge system. The base and catalyst are automixed by forcing the materials through a disposable mixing tip. Insure Lite Automix is light cured at wavelength range of 400 – 500 nm and self-cures in the absence of light.

Please note: Although Insure Lite Automix is dual-cure, it must be light-cured to ensure quality.

COMPOSITION

Insure Lite Automix contains UDMA and BIS-EMA (40%). The filler contains barium glass (59% by weight and 36% by volume). The particle size of the fillers range from 0.1 – 3 micron. Initiators and stabilizers are additional ingredients (< 1%). It contains benzoyl peroxide.

INDICATIONS FOR USE

Cementation of ceramic or composite crowns, inlays, and onlays.

APPLICATION

① Prepare tooth.

② Etch tooth structure with 37% phosphoric acid followed by thorough rinsing. Use a 4th or 5th generation dentin/enamel adhesive or an advanced adhesive such as Cosmedent's Complete™ on the etched tooth and the restoration following manufacturer's instructions.

③ Porcelain restorations are typically etched and silanated when delivered from the lab. If the restoration require re-etching, a 4 minute etch with hydrofluoric acid followed by rinsing and silane treatment is required.

④ It is very important to clear or bleed the syringe prior to applying the mixing tip. Gently press the dispenser lever until both components (base and catalyst) begin to flow out evenly onto a mixing pad. To apply the mixing tip, push the tip completely down until the notch of the mixing tip is aligned with the notch on the housing of the cartridge/ syringe. Secure the mixing tip in place by gripping the colored base, (not the clear mixing tip), and by turning it 1/4 turn clockwise. For each application, place a new mixing tip on the syringe. For each new tip, bleed a small pea size amount of material prior to use. Keep the tip immersed in material while dispensing to avoid bubbles.

⑤ Maximum working time is 3.5 ± 0.5 minutes. Seat the restoration using gentle finger pressure before the cement begins to set. Remove excess cement with an IPC carver except for a narrow string of material at the margins.

⑥ Light cure ceramic and composite restorations for 15 seconds to tack restoration in place.

⑦ Remove any excess cement and cover margins with glycerin to eliminate air inhibited cement at the margins.

⑧ Finally, light cure ceramic and composite restorations for 60 seconds including margins.

⑨ In the self-cure mode, the final set is reached 5.5 minutes after the beginning of the mix.

TIMING FROM THE BEGINNING OF THE MIX

Working time:

3.5 ± 0.5 minutes

Self-cure setting time:

5.5 minutes

final light cure:

60 seconds

HYGIENE / DISINFECTION

In order to prevent polymerization or clogging, the dual-syringe mixing tip should be left in place until the next application. If used intraorally,

wipe mixing tip with disinfectant prior to storage. The Automix-Dispenser is reusable after disinfection. Disinfection can be carried out using a commercially available disinfecting solution.

STORAGE

- Once opened, do not refrigerate.
- Store at room temperature, approximately 39 – 75°F (4 – 24°C).
- Shelf life: see information on syringes and/or packaging.
- Keep away from sunlight
- Keep out of reach of children.
- Do not use after the expiration date.

R IMPORTANT PRODUCT

INFORMATION

Federal (U.S.) law restricts this device to sale by or on the order of a dentist.

NOTE

This product was developed for use in the dental sector and should be used in accordance with the instructions for use. The manufacturer does not accept liability for damage caused by its use for any other purpose. The user is also obliged to accept responsibility for checking before use that the material is suitable and can be used for the intended purpose, particularly if this is not listed in the instructions for use.

DISPOSAL INSTRUCTIONS

Dispose of contents/ container in accordance with local regulation.

CONTRAINDICATIONS

Insure Lite Automix is contraindicated if a patient is known to be allergic to any of the ingredients.

SIDE EFFECTS

May cause irritation in sensitive patients.

INTERACTIONS

Do not use in combination with eugenol containing materials, as these may impede the polymerization process.

NOTICE

During time-consuming procedures, avoid overexposure from the operatory light to protect against premature polymerization of the material. The light curing unit must be maintained and calibrated according to the manufacturer's specifications. Serious incidents must be reported to DeltaMed GmbH, the responsible importer and the responsible authority.

WARNINGS

Insure Lite Automix contains Urethane dimethacrylate and Dibenzoyl peroxide. May cause an allergic skin reaction. Wear protective gloves. In case of contact with the eyes, rinse immediately with plenty of water and seek medical attention.

PACKAGING

8 g syringes (includes 10 mixers)

SHADE

Insure Lite Automix is available in the following shades: Clear

Canada Site Licence Holder:
CLINICAL RESEARCH DENTAL
167 Central Ave, Suite 200
London, Ontario, Canada N6A 1M6
Establishment License No. 666

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Cosmedent® Inc.