EN

COSMEDENT™ www.cosmedent.com | 800.621.6729

Prevue® Try-In Gel System

INSTRUCTIONS FOR USE

DESCRIPTION

Prevue® water-soluble try-in gels take the guesswork and messiness out of porcelain and indirect resin veneer color evaluation. It is exactly color-matched to all 7 shades in the Insure® Cementation System.

Veneers are loaded with Prevue Try-in Gel, and placed over all prepared teeth, permitting complete visualization of the veneer by the patient.

Prevue Try-in Gels are easily removed from veneers and tooth structure with a water spray.

SHADES

Prevue Try-in Gels are available in the following shades, color-matched to the Insure shades:
Clear, Yellow Red Light, Yellow Red Universal, Grey Light, White Opaque, Yellow Opaque, Pink Opaque

COMPOSITION

Prevue Try-In Gel consists of propylene glycol, glycerin, water, acrylic polymer and pigments. It is pH neutral.

INDICATIONS

(Areas of application)
– Water soluble try-in veneer cement system.

DISPENSING PREVUE TRY-IN GEL

Place the disposable tip over the syringe opening and screw it on firmly before dispensing Prevue Try-In Gel. This will provide controlled placement into the veneer shell without excess Try-In Gel extruding from the syringe causing material waste. Throw away tip after use and replace the cap on the Try-In Gel syringe.

APPLICATION

Before treatment, tooth shade should be taken using a standard shade guide. Keep teeth moist when taking the shade.

- Select the appropriate Prevue Try-In Gel shade or blend for trial evaluation.
- Fully load the previously etched internal surface of each veneer with Prevue Try-in Gel.
- Clean and dry all teeth receiving the veneers but do not etch the teeth or condition the dentin.
- Place each veneer on the recipient tooth and seat completely with light pressure. Wipe away any excess Prevue Try-in Gel from the margins.
- Place the patient in an upright position and evaluate color before a large mirror under multiple lighting conditions, e.g. daylight, office lighting, etc.
- If the color is favorable, select the corresponding Insure resin cement or resin cement mix, e.g. Clear Prevue Try-In Gel would require Clear Insure resin cement.
- Remove the veneers from each tooth. With a vigorous water spray, rinse the Prevue Try-In Gel from the veneers and the teeth.

HYGIENE/DISINFECTION

– Disinfection of the syringe can be carried out using a commercially available disinfecting solution.
– All instruments used should be thoroughly washed and autoclaved.

STORAGE

– Store at 39 – 75°F (4 – 24°C).
– Shelf life: see information on syringes and / or packages.
– Keep away from sunlight
– Keep out of reach of children.
– Do not use after the expiration date.

IMPORTANT PRODUCT INFORMATION

Federal (U.S.) law restricts this device to sale by or on the order of a dentist.

Note: This product was developed for use in the dental sector and should be used in accordance with the instructions for use. The manufacturer does not accept liability for dam- age caused by its use for any other purpose. The user is also obliged to accept responsibility for checking before use that the material is suitable and can be used for the intended purpose, particularly if this is not listed in the instructions for use.

DISPOSAL INSTRUCTIONS

Dispose of contents/ container in accordance with local regulation.

CONTRAINDICATIONS

Placement of Prevue Try-In Gel is contraindicated if a patient is known to be allergic to any of the ingredients.

SIDE EFFECTS

May cause irritation.

INTERACTIONS

None

NOTICE

Serious incidents must be reported to DeltaMed GmbH, the responsible importer and the responsible authority.

WARNING

Prevue Try-in Gels contain glycerin and propylene glycol and may cause irritation in sensitive persons. In case of contact with the eyes, rinse immediately with plenty of water and seek medical attention if irritation persists.

PACKAGING

Prevue 2.5 g syringes (includes 3-19 gauge tips)

Canada Site Licence Holder:
CLINICAL RESEARCH DENTAL
167 Central Ave, Suite 200
London, Ontario, Canada N6A 1M6
Establishment License No. 666

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Cosmedent® Inc.